
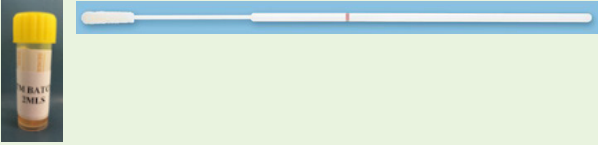
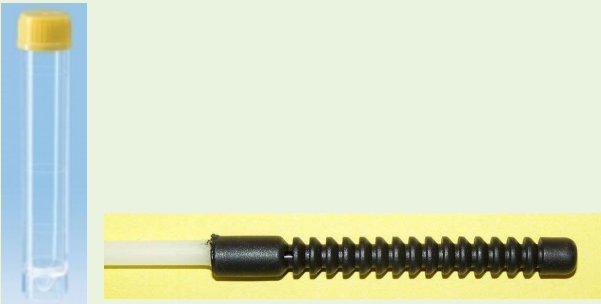


SWAB COLLECTION GUIDE

SWAB DESCRIPTION	USE	INSTRUCTIONS
<p>Blue Top Plastic Shaft Swab (Amies Transport Medium) Order Code:15924</p> 	<p>Bacterial & Yeast Microscopy, Culture & Sensitivity (M,C&S)</p> <p>(Not suitable for PCR & Fungal culture)</p>	<p>After collection, place blue top swab into transport medium; discard white top. If both M,C &S and PCR are required an appropriate swab for PCR must also be collected. Store at 2 -8°C. Needs to be processed within 12-18 hours for culture</p>
<p>White Top Plastic Shaft Swab (No Medium/Dry Swab) Order Code: 15923</p> 	<p>Suitable for PCR & Fungal Testingculture tube.</p>	<p>After collection, place white top swab back into dry tube. Store at Room Temp for Fungal Testing or 2 - 8°C for PCR-</p>
<p>FLOQSwabs™ & PBGS Viral Transport Medium (VTM) Streile Nylon brush swab &VTM vial To order contact Gribbles VETLAB</p> 	<p>Viral Culture and/or Viral PCR:</p> <p>e.g. Vesicular Disease (FMD), AI,NDV, Hendra Virus exclusion</p>	<p>Store VTM vials at or below -20°C. Unused VTM vials that have been defrosted can be re-frozen and re-used.</p> <p>After collection, break off swab tip using the molded break point into the vial of VTM. Store at 2 - 8°C</p>
<p>Green Top Plastic Shaft Swab (Viral Transport Medium) To order contact Gribbles VETLAB</p> 	<p>Viral Culture and/or Viral PCR:</p> <p>e.g. Vesicular Disease (FMD), AI,NDV, Hendra Virus exclusion</p>	<p>Prior to use: Store Swab pack at 5-25°C. Please check the expiry date.</p> <p>After collection, place green top swab into transport medium. Store at Room Temp.</p>
<p>Tricamper™ & PBS Black corrugated sampling device To order contact Gribbles VETLAB Price: \$5/swab</p> 	<p>Bovine Infertility & Abortion Testing</p> <p><i>Campylobacter fetus subsp venerealis and Trichomonas foetus</i></p>	<p>Move the tool back and forth to sample vaginal mucus or preputial smegma, cut off the black head of the Tricamper with side cutters into PBS or media. For more detailed instructions contact Gribbles VETLAB. Store at 2 -8°C.</p>